

BRON-AIRE™ VOLUME SPACER



BRON-AIRE'S™ PRINCIPAL FUNCTION

All patients who require respiratory therapies for treatment of respiratory problems, should use **BRON-AIRE™** to get the inhaled medication to the bronchial tubes (lungs).

Never use spray without spacer. Using **BRON-AIRE™** gives better results than other traditional therapies.

WHO NEEDS BRON-AIRE™?

All patients, kids, adults and elderly people, who their doctor has prescribed respiratory therapies for their respiratory problems.

BRON-AIRE™ must be used for all patients who suffer from: Asthma, acute and chronic bronchitis, COPD, smoker's cough, etc.

BENEFITS YOU RECEIVE WHEN USING BRON-AIRE™

- Better results than any other therapy and at lower cost
- It can be used in crisis cases
- Quick results
- It can be used at any age
- Portable
- Fits any aerosolized medication
- No electrical energy required
- Available in all pharmacies in the country (specifically in El Salvador)

* For more information, consult your doctor. Medicine NOT included.

ADVANTAGES

1. **BRON-AIRE™** doesn't need respiratory strength. Spacers with valves need strength and patients with respiratory problems have reduced this force, thus less medicine can reach the bronchial tubes-lungs.

2. Results in crisis may be lower if the patient uses Volume Spacers with valves, because they need to make an extra patient's respiratory effort
3. **BRON-AIRE™** doesn't have a valve that hinders the transfer of the medicine to the bronchial tubes - lungs
4. **BRON-AIRE™** is easy to clean; meanwhile the ones that have a valve, accumulate dirt and thus can be inhaled by the patient.
5. The price of the Volume Spacers with valves is very high, without providing further benefits to patients
6. **BRON-AIRE'S™** price is accessible to all patients and to all health institutions

How to clean correctly your BRON-AIRE™ Volume Spacer?

1. Put 1 liter of water at room temperature in a container and add 1/2 ounce of liquid soap or detergent
2. Immerse all the accessories of **BRON-AIRE™** and move them into the water, leaving them for 15 minutes
3. Let the accessories dry on their own about 1 hour or use a hair dryer for a faster method
4. Assemble **BRON-AIRE™** and their accessories in the right way
5. Ready for use

RECOMMENDATIONS FOR TAKING CARE OF YOUR BRON-AIRE™

- Wash your **BRON-AIRE™** and its accessories before the first use
- Don't use bleach or alcohol for cleaning or washing
- Don't wash with hot water
- For more hygiene, wash your **BRON-AIRE'S™** every week

*Following these indications and recommendations, we avoid plastic static and help to enlarge the shelf life of the Volume Spacer.

SHELF LIFE

BRON-AIRE'S™ Volume Spacer shelf life is approximately 1 year, depending on factors like: weekly or monthly cleaning and personal care.

Ref.: 1 Pierart et. al. Washing plastic spacers in household detergent reduces electrostatic charge and greatly improves delivery. Eur Respir J 1999; 13:673-8.2. Devadason. Recent advances in aerosol therapy for childrens with asthma. Journal of aerosol medicine.